DE Massage-Fußwärmer Gebrauchsanweisung	2	IT Scaldapiedi con massaggio Istruzioni per l'uso.....	26
EN Massaging foot warmer Instructions for use	8	TR Masaj ayak ısıtıcısı Kullanma Talimatı.....	32
FR Chauffe-pieds massant Mode d'emploi.....	14	RU Грелка для ног Инструкция по применению.....	38
ES Calientapiés con masaje Instrucciones de uso.....	20	PL But rozgrzewający z funkcją masażu Instrukcja obsługi.....	44

Inhalt

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Contents

1. Included in delivery..... 9	4.4 Massage settings/heat settings..... 11
1.1 Unit description 9	5. Cleaning and maintenance..... 11
2. Important instructions 9	6. Storage 12
3. Intended use..... 10	7. Disposal..... 12
4. Operation..... 10	8. What if there are problems? 12
4.1 Initial use..... 10	9. Technical data 12
4.2 Switching on 11	10. Warranty / service..... 12
4.3 Switching off..... 11	

Signs and symbols


	Read instructions!		Do not wash
			Do not bleach
	Do not insert needles!		Do not tumble dry
			Do not iron
	Only use in dry rooms		Do not dry clean
IPX1	Drip proof		Must not be used by very young children (0-3 years old)
	Dispose of packaging in an environmentally friendly manner		
	WARNING; Warning instruction indicating a risk of injury or damage to health		
	IMPORTANT; Safety note indicating possible damage to the unit/accessory		
	NOTE; Important information		

1. Included in delivery

- 1 Foot warmer with fleece lining
- 1 Mains part
- 1 Instructions for use

1.1 Unit description

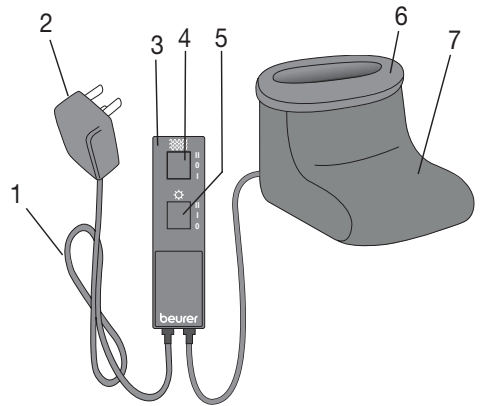- 1. Cable
- 2. Mains part
- 3. Controller
- 4. Sliding switch for massage settings (0-1-2)
- 5. Sliding switch for heat settings (0-1-2)
- 6. Fleece lining
- 7. Foot warmer

2. Important instructions

Retain for future reference

WARNING

- Non-observance of the following notes may result in personal injury or material damage (electric shock, skin burns, fire). The following safety and hazard information is not only intended to protect your health and the health of others, it should also protect the product. For this reason, pay attention to these safety notes and include these instructions when handing over the product to others.
- This foot warmer must not be used by people who are not sensitive to heat or by vulnerable people who may not be able to react to overheating (e.g. diabetics, persons with skin changes related to illness or persons with scarred skin in the area where the pad is to be applied, after taking any pain-relieving medication or alcohol).
- This foot warmer must not be used by very young children (0-3 years old) as they are unable to respond to overheating.
- This foot warmer must not be used by young children (3-8 years old) unless the controller has been set by a parent or responsible adult and the child has been given adequate instructions on how to use the foot warmer safely.
- This foot warmer may be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised and have been instructed on how to use the foot warmer safely, and are fully aware of the consequent risks of use.
- Children must not play with the foot warmer.
- Cleaning and user maintenance must not be performed by children unless supervised.
- This foot warmer is not designed for use in hospitals.

- This foot warmer must not be used to keep animals warm.
- Do not use this foot warmer with shoes.
- Do not insert needles.
- Only use in dry rooms.
- Do not use if wet.
- Use only the original power supply provided Type:
EU: KL-AD3060VA
GB: RSS1006-240120-W3U-B-C
AU: RSS1006-240120-W2A-B
- The electrical and magnetic fields emitted by this foot warmer may interfere with the function of a pacemaker. However, they are still well within the limits: electrical field strength: max. 5000 V/m, magnetic field strength: max. 80 A/m, magnetic flux density: max. 0.1 millitesla. Please consult your doctor and the manufacturer of your pacemaker before using this foot warmer.
- Do not pull, twist or make sharp bends in the cables.
- Please check this foot warmer frequently for signs of wear and tear or damage.
If any such signs are evident, if the foot warmer has been used incorrectly or if it no longer heats up, it must be checked by the manufacturer before being switched on again.
- If the cables of the foot warmer are damaged, they must be replaced by the manufacturer, its customer service department or a similarly qualified person to avoid any risk.
- Always observe the notes on operation (section 4), cleaning and maintenance (section 5) and storage (section 6).
- If you should have any questions about using our devices, please contact Customer Services.

3. Intended use

WARNING

This product is intended for warming and massaging human feet. Do not use the foot warmer with shoes, as they reduce your perception of heat and dirty the foot warmer. This foot warmer is not designed for use in hospitals or commercial use. The following groups in particular must not use the device: infants, small children, vulnerable persons, persons with low sensitivity to heat or animals. Otherwise, there is a risk of injury.

4. Operation

4.1 Initial use

- Connect the mains part cable to the foot warmer controller.
- Insert the mains part into the socket.

4.2 Switching on

The fastest way to warm up the foot warmer is to initially select the highest heat setting (setting 2). The setting can be reduced later if necessary.

The foot warmer can, of course, be used at any time, even when it is switched off: its fleecy fabric will warm the feet if there is no socket nearby.

4.3 Switching off

To switch the foot warmer off, set the sliding switch for massage and heat settings to the setting "0" (OFF).

NOTE

If the foot warmer is not in use, set the sliding switch to "0" (OFF). Disconnect the mains part from the foot warmer. Remove the mains part from the socket.

4.4 Massage settings/heat settings

- Massage only:

Setting I (low): relaxing massage

Setting II (high): invigorating massage

The sliding switch for the heat settings is in the "0" position.

- Heat only:

Setting I: minimum heat

Setting II: maximum heat

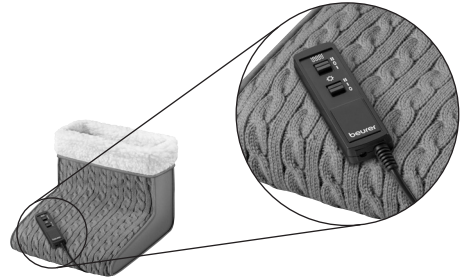
The sliding switch for the massage settings is in the "0" position.

- Massage with heat:

Massage and heat setting freely selectable.

- No massage or heat:

Both sliding switches are set to "0".

WARNING

If the foot warmer is being used over several hours, we recommend that you select the lowest heat setting on the controller to avoid overheating the feet, which may lead to burns to the skin.

5. Cleaning and maintenance

WARNING

Before cleaning, always remove the mains part from the socket first.

Then disconnect the mains part cable from the foot warmer controller.

Otherwise, there is a risk of damage.

IMPORTANT

- The controller must never come into contact with water or other liquids, as this may cause damage

To clean the controller, use a dry, lint-free cloth. Do not use any chemical or abrasive cleaning agents.

Small marks on the foot warmer can be removed with a damp cloth and, if necessary, with a little liquid detergent for delicate laundry.

IMPORTANT

- Please note that the foot warmer must not be washed, chemically cleaned, wrung out, tumble dried, put through a mangle or ironed. Failure to observe this may result in damage to the foot warmer.

The removable fleece lining can be washed by hand.

Use a delicate laundry detergent and measure it out according to the manufacturer's instructions.

WARNING

- Only use the foot warmer again once the foot warmer and fleece insert have fully dried. Otherwise, there is a risk of damage.

6. Storage

If you do not plan to use the foot warmer for a long period of time, we recommend that you store it in the original packaging.

IMPORTANT

Allow the foot warmer to cool down before storing. Failure to observe this may result in damage to the foot warmer. To avoid damaging the foot warmer, do not place any objects on top of it whilst storing.

7. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

8. What if there are problems?

Problem	Cause	Solution
<p>The foot warmer does not warm up or the massage function cannot be activated while:</p> <ul style="list-style-type: none"> – the mains part is connected to a working socket. – the foot warmer controller is connected to the mains part cable. – both sliding switches are set to setting 2 (heat + massage). 	<p>The foot warmer is faulty.</p>	<p>Send the foot warmer for servicing.</p>

9. Technical data

See type plate on the foot warmer controller.

10. Warranty / service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller's statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.
German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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